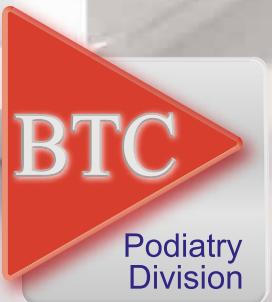
INGENii

Interactive Podiatry Unit

Innovative Medical Solutions

WHO WE ARE | QUI NOUS SOMMES | CHI SIAMO

THE COMPANY

BTC Medical Equipment is specialized in developing and manufacturing Multispecialty Examination and Procedure Chairs, Podiatry Units and Chairs, Gait Analysis and Orthotics Equipment.

BTC is committed to the continuous research of innovative technical solutions for the Professionals; the range of equipment offered is based on and follows rigid guidelines: latest technology, perfect work-flow ergonomics, reliability, design and highly intuitive and easy to operate by the Professional.

The Company is directly involved through all development, manufacturing, assembly and quality control, shipment, installation and after-sale service phases thanks to its vast network of Distributors and Agents.

In the Company premises located in IMOLA BTC has set up a permanent Show-Room, a Training Area for theory-practice sessions and the interna Service Department.

A well structured company organization that has enabled BTC to grow at national and international level keeping the highest quality standards and constant innovation of its range of products.

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"All BTC Medical Equipment is designed, developed and manufactured in Italy"

HOW WE WORK | COMMENT NOUS TRAVAILLONS | COME LAVORIAMO

R&D

BTC R&D Team develops internally all the equipment and devices for the Professionals and Clinics, always combining modern and essential design to the latest and most innovative technical solutions.

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R&D
R&D
R&D

BTC manufactures a wide and complete range of Units, Chairs and Doctor Stools in its premises located in Imola. Every single production phase is planned and performed according to the most rigid quality control criteria that have enabled BTC to obtain the ISO:13485 Quality Certificate by CISQ/IMQ

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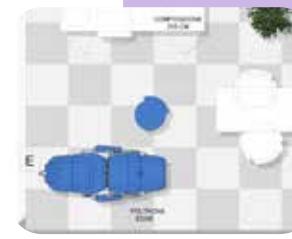
PRODUCTION
PRODUCTION
REALIZZAZIONE


BTC will design your Ambulatory in order to allow for the best work-flow and ergonomics while guaranteeing maximum comfort to the Patient, according to the space availability and your different set up needs as a Professional.

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PROJECT
PROJET
PROGETTO

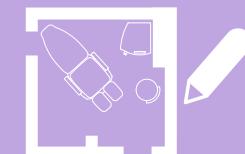INSTALLATION
INSTALLATION
INSTALLAZIONE

All BTC equipment is "Plug-in & Work" easy to set up and ready to use with very few and easy installation steps. BTC guarantees a highly professional installation service thanks to its vast network of Distributors and Agents, and a product specialist will train the Ambulatory staff on the correct use of the installed equipment.

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LOCATION LOCATION LAZIONE

TRAINING FORMATION FORMAZIONE

BTC is committed to continuous training and information of all its Customers; the Company can rely on the cooperation with prominent experts in each medical discipline and often involved in training sessions on the specific equipment and different operating methods.

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BTC considers Warranty a professional duty and mandatory towards all its Customers, reason being the Company extends guarantee of the equipment up to 2 Years on Electronic Parts and up to 5 Years on Mechanic Parts.

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All Customers can rely on a punctual and qualified after-sale assistance thanks to the in-house Service Department. You will be able to request a Preventive Maintenance and Courtesy Service to our Distributors and avoid any unexpected interruption of your professional activity.

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INGENii

Interactive Podiatry Unit

INGENii is the latest Podiatry Unit developed by BTC, equipped with an interactive operating system and provided with a level of technological innovation never reached before.

INGENii is the result of innovative technical and engineering solutions that guarantee maximum ergonomics and functionality to professionals during all Podiatry treatments.

BTC owns a registered patent on its Unit

INGENii is the result of a state of the art design, subject to community registration, guaranteeing a modern and essential style in every single detail.

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INGENii is a modern Podiatry Unit, able to combine intuitive technology of use, with innovative ergonomic solutions ensuring exceptional comfort for the operator.

The Unit is made up of two cabinets - a technical compartment and a drawers storage - which can be oriented and positioned according to the available space and following the most correct positioning of the operator.

The instrument tablet and the 10-inch touchscreen control panel are connected to a triple articulated arm allowing the best operational ergonomics.

The control panel with touchscreen has been designed to guarantee a whole range of multimedia functions able to support the daily work of the professional, allowing the management of all the functions of the Unit, as well as the display of alarms and information.

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INGENii is equipped with a latest generation 10-inch (capacitive) touch screen integrated in a light and solid case made in anodized aluminum. The software, developed by BTC on a Linux basis, is able to manage all daily activities of the professional and it is designed to provide a whole series of multimedia contents useful for his work.

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When activated, **INGENii** allows you to choose and name the operator with which you connect. The BTC operating system is able to create and customize up to 5 different operators, including all the desired settings referring to all individual accounts.

INGENii can be connected to both Wi-Fi and LAN and control all functions of the patient chair, thanks to the XBee RF module.

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FAVORITES bar customizable by the Operator allows a quick recall of the most frequently browsed sections

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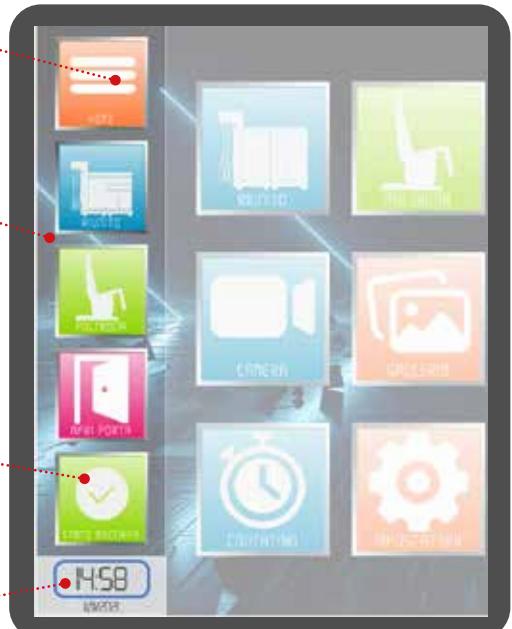
Some welcome icons allow you to use all the functions of the INGENii Unit

- PODIATRY UNIT
- CHAIR
- OPERATING LAMP
- UV DRAWER
- MONITOR
- CAMERA
- GALLERY
- USER MANUAL
- DOOR OPENER
- AGENDA
- SERVICE HOUR COUNTER
- SETTINGS
- LANGUAGE

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From the "UNIT" APPLICATION it is possible to manage:

- dynamic instruments (Spray micromotor, Suction micromotor, turbine and syringe)
- the orientation of the tablet
- detachment of the drawers storage
- lighting
- the UV compartment
- the camera or the external screen, if they are present.

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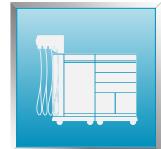
PODIATRY UNIT
UNITÉ DE SOIN
RIUNITO

INGENii is equipped with the mm Spray BTC Ghibli micromotor.

The micromotor is brushless and is suitable for LED light. The maximum rotation speed is 40,000 rpm, the handpiece connection is ISO type and is guaranteed 24 months by BTC.

From the touchscreen, it is possible to adjust the speed, as well as to store and recall 8 memorized speeds, then activate / deactivate the spray function, then reverse the rotation of the bur or turn on/off the handpiece light. According to the selected speed, the correct diameter of the most suitable bur to use is displayed and highlighted.

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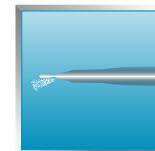
**SPRAY MM
MM SPRAY
MM SPRAY**

INGENii is equipped with the mm Aspi BTC Air.

The brushless micromotor is equipped with a handpiece integrated into a suction system. The maximum rotational speed is 40,000 rpm. 24 months warranty provided by BTC.

From the touchscreen, it is possible to adjust the speed, as well as to store and recall 8 memorized speeds, then activate / deactivate the suction engine, then reverse the rotation of the bur. According to the selected speed, the correct diameter of the most suitable bur to use is displayed and highlighted.

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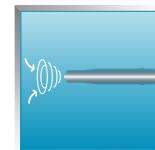
**SUCTION MM
MM ASPIRATION
MM SPRAY**

INGENii is designed for the installation of a Bravia L (equipped with Led) or Thalia turbine.

From the touchscreen, it is possible to adjust the speed, as well as to store and recall 8 memorized speeds, then activate / deactivate the spray function or switch on the OFs.

Bravia L: high speed turbine equipped with ceramic ball bearings, triple spray, bright lighting and ergonomic structure. Easy to clean surface.

Thalia: single spray and high torque turbine and fixed connection. The turbine is lightweight, easy to clean.

OPTIONAL

INGENii is equipped with a 3-function syringe: air, water and spray. The tip and the handle are easily removable to ensure perfect disinfection and sterilization in autoclave (135 °).

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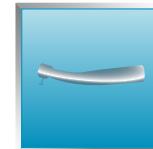
TURBINE TURBINE TURBINA

SYRINGE SERINGUE SIRINGA

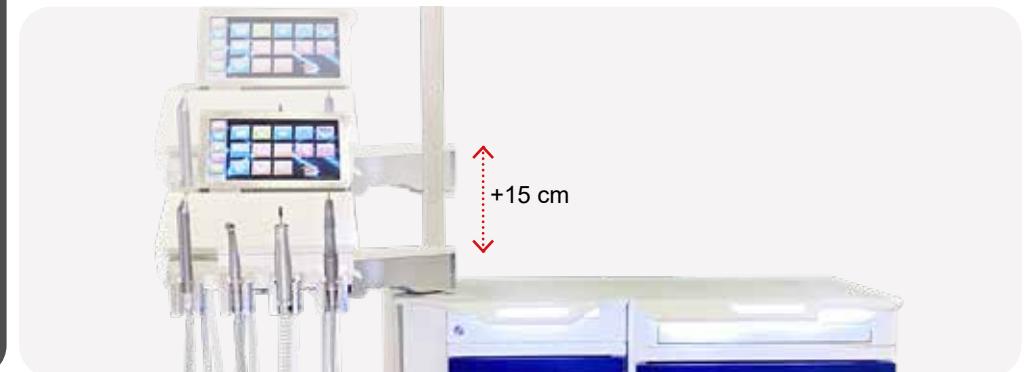
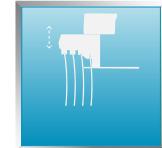
INGENii is equipped with a motorized arm allowing a 15 cm height excursion of the instrument's tablet. Run by a linear actuator, it guarantees perfect ergonomics.

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INGENii is equipped with a pair of electromagnets keeping the two sections of the Cabinet together, allowing them to be separated when this function is activated via touch screen. To rejoin the Cabinet again, simply move the drawer's storage to the technical compartment: the magnetic system will automatically attract both parts.

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STORAGE DETACHMENT
SEPARATION ELEMENT À
TIROIR
DISTACCO CASSETTIERA

Thanks to the XBee radio frequency module, **INGENii** is able to control the Chair connected to the Unit, via the touch screen. From the panel it is possible to control single movements of the Chair, lock and recall the 7 memorized positions. During any single Chair movement, the touchscreen shows the operator the corresponding 3D graphics of that movement. It is possible to combine both the EDGE and MOTION range of Chairs.

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CHAIR
FAUTEUIL
POLTRONA

LAMP
LAMPE
LAMPADA

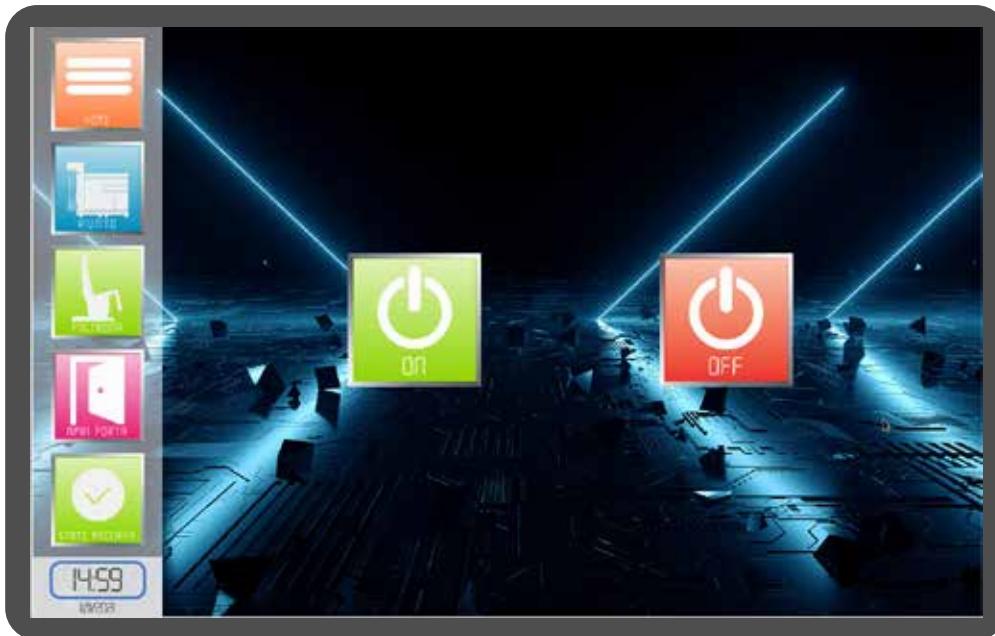
INGENii is equipped with a 60 cm support arm for lighting. The configuration is to be completed with a Lamp at choice among BTC range. The Lamp will be powered by an integrated system into the Unit and will be controlled via the touch screen panel.

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INGENii is equipped with a UV drawer with LED light. The touch screen panel allows the UV light to be turned on and off. The UV drawer is approximately 15 cm high. It is also equipped with aluminum trays. The engraving on the front drawer allows the Operator to check the ignition of the UV compartment at a glance.

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UV DRAWER
TIROIR UV
VANO UV

INGENii can be equipped with a camera and/or external monitors to be connected via USB. Both USB outputs can be enabled / disabled by the touch screen panel.

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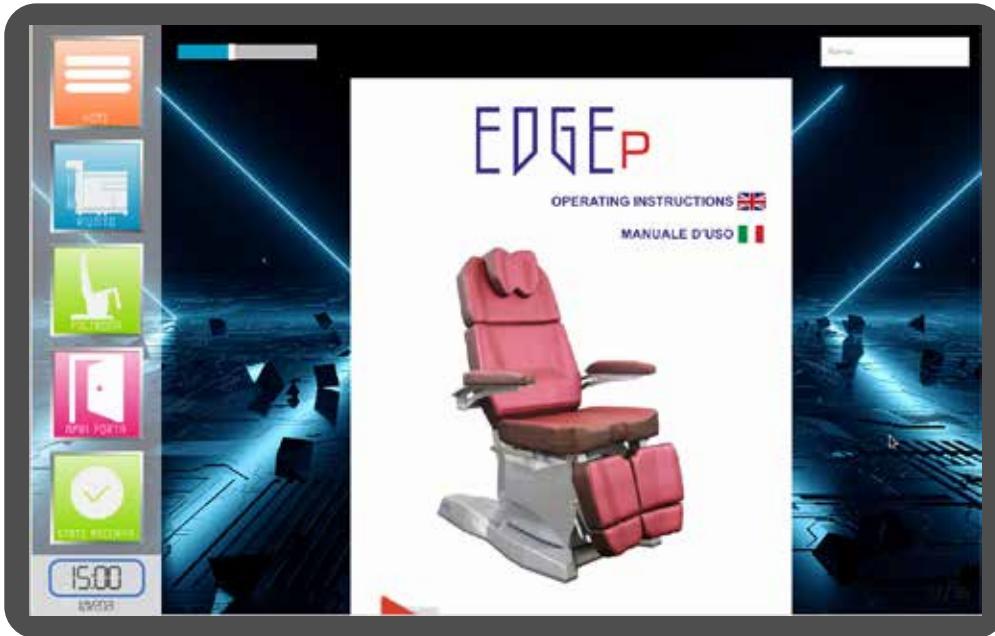
CAMERA
CAMERA
CAMERA

MONITOR
ECRAN
MONITOR

INGENii allows the Operator to archive images or videos if they are present in the USB key. The Gallery section will provide a view of all archived files.

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GALLERY
GALERIE
GALLERIA

INGENii allows the Operator to consult all the User Manuals (Ingenii and combined Chair User Manual). The User Manuals section also presents videos for proper and correct use of the Unit as well as its routine maintenance.

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USER MANUALS
MODE D'EMPLOI
MANUALI

INGENii allows a door to be opened with an electric lock, if it is connected to the Unit.

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**DOOR OPENER
GACHE
APRI PORTA**

INGENii provides an Agenda allowing the Professional to directly manage any appointments from the Unit. The calendar is available both offline and synchronized with Google Calendar. A reminder message is displayed 10 minutes before any appointment at the top of the touchscreen, regardless of which page is open.

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**CALENDAR
AGENDA
AGENDA**

31 Calendar

INGENii records hours worked by the Unit and main components into the Counter section. The number of hours is useful to schedule any maintenance activity.

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INGENii can be set up in 5 languages. All texts and icons are translated into the selected language.

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**LANGUAGE
LANGUE
LINGUA**

INGENii is equipped with a Setting panel, providing the Unit with a high level of customization.

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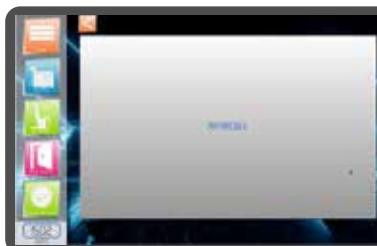
SETTINGS PARAMÈTRES IMPOSTAZIONI

Unit name: customize the name of **INGENii**, visible on the Homepage of the touch screen

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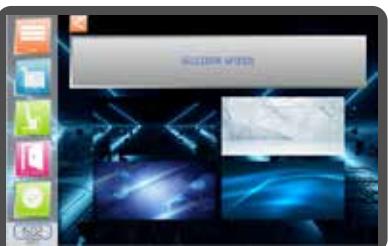
Favorite setting: to customize the icons of the Favorites sidebar. Simply drag the icon you want to be displayed.

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Date and Time: Allows to set the date and time in case the unit is offline. When connected to the LAN or Wi-Fi network, the date and time are automatically updated by the system.

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Courtesy lights adjustment: allows to adjust the intensity of the courtesy lights.

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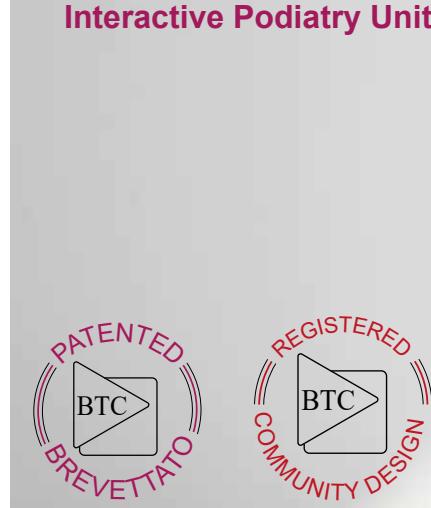
Text settings: allows to change the text format according to the Operator's choice.

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INGENii Interactive Podiatry Unit

INGENii is manufactured with a steel structure painted with epoxy powders, able to ensure the maximum resistance over time.

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INGENii is equipped with powder coated steel front drawers with Push Pull opening system. The absence of handles provides easy cleaning and disinfection. The front drawers can be customized with a color at choice among the wide range of RAL colors available.

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The front drawers are equipped with Push-Pull guiding rails with a stabilizer bar. Opening by a simple pressing on any spot of the front drawer.

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The two lower drawers are also available in a Push-Pull version provided with an impact-resistant stainless steel reinforcement plate and open/closing system by foot.

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Hot bending Corian® worktop. Corian® meets the stringent design requirements for medical environments. Easy to clean and maintain. LGA certified for its hygienic properties (stability against bacteria). Smooth, nonporous and no welding surface, extremely durable overtime and impact resistant.

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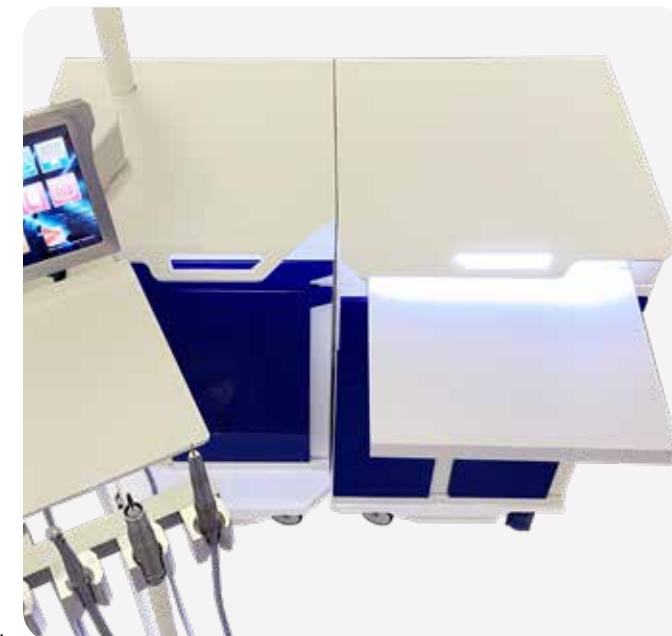
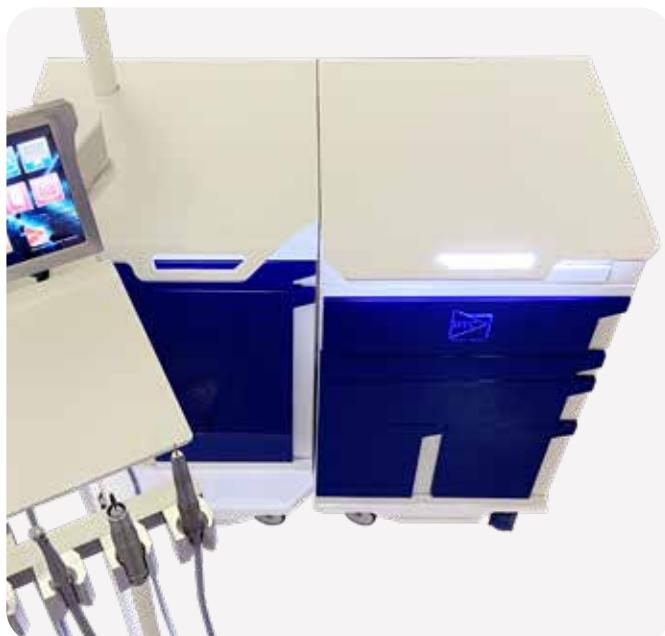
The courtesy light is useful for illuminating the inside compartment of the drawer, the light intensity is adjustable by the Settings section of the touch screen.

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Below the drawers storage worktop, a removable Corian shelf provides the Operator with a desk, where to place the Wi-Fi keyboard or mouse in combination with the touch screen.

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The base of the Cabinet consists of a technical compartment and a drawer storage.

The two sections are independent and can be positioned at will, in order to ensure the maximum ergonomics to the Operator.

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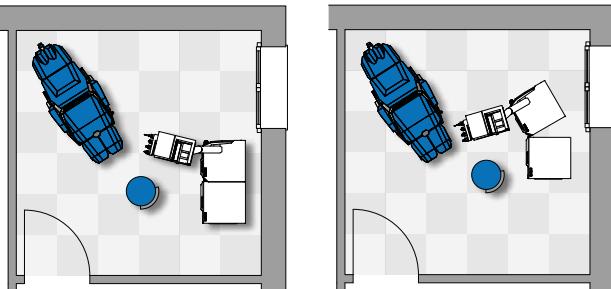
The two Cabinets are adjustable from 0 to 30 ° allowing the Operator to adapt the Unit to the available area. The rotation is guaranteed by steel bearings. The two sections remain attached by means of two electromagnets which can be released by the corresponding function activated by the touch screen.

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
The instrument tablet is mounted on a support arm with triple articulation in order to provide the Operator with a perfect orientation.

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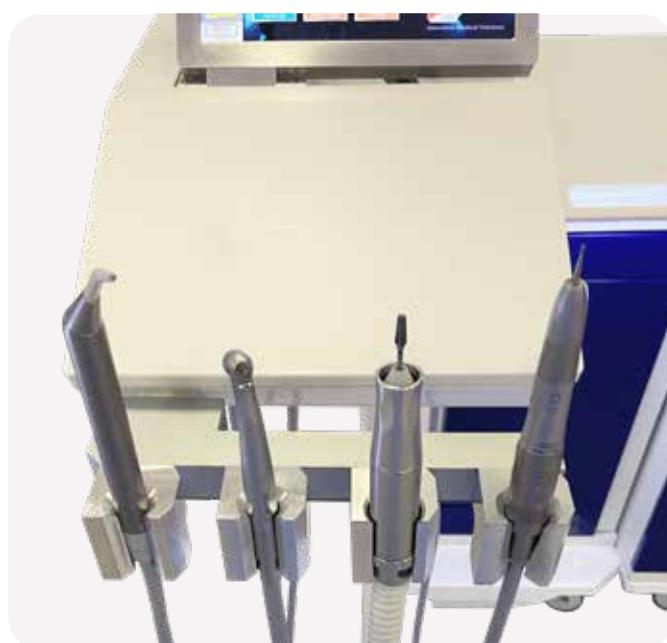The dynamic instruments are housed in an aluminum case equipped with an infrared presence sensor. When the instrument is held out, the integrated system automatically opens the instrument function page.

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The instruments are provided on an adjustable handle (0-20°) to allow the Operator to adjust the angle of the instruments.

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Below the shelf, a removable tray is available with a small aluminum tray to store the tools during treatments. The tray can be removed if necessary.

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The touch screen is adjustable and removable for perfect positioning. The panel is integrated into a sturdy and stylish anodized aluminum frame equipped with a side handle to allow handling. The frame is fixed to a hinge and a linear guide allowing smooth movements.

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On the side of the tablet, 2 USB sockets are available for plugging in USB keys

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The door hosts a compartment for the suction motor filter and the distilled water tank for the spray liquid. Easy access for the operator in charge of ordinary maintenance.

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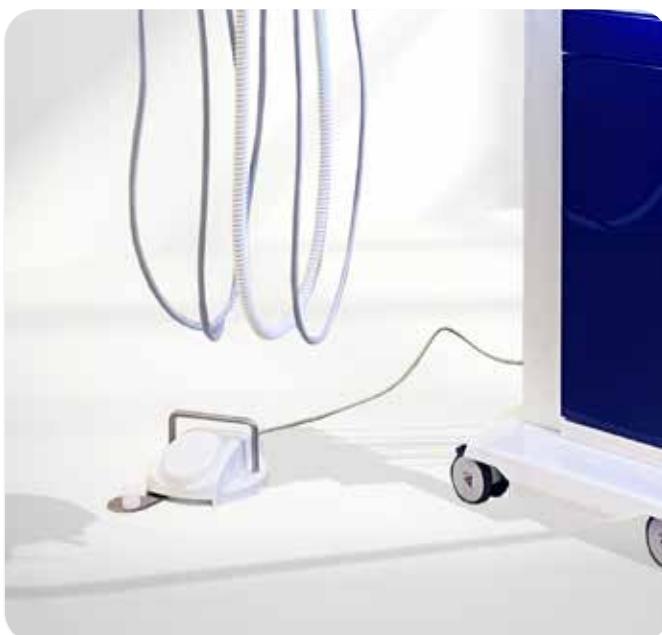
Brushless suction engine LONG LIFE integrated inside the cabinet.

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Chip blower function is activated by pressing the lever. Spray is activated by pressing the lever while the instrument is rotating.

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Integrated compressor inside the lower drawer of the storage, equipped with ventilation and soundproofing. It provides air jet in support of pneumatic and electric devices (40 l/min.).

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codE description
3620146 BRAVIA L

**BRAVIA L**

High speed turbine equipped with ceramic ball bearings, triple spray, bright lighting and ergonomic structure. Easy to clean surface.

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code description
3620148 THALIA

**THALIA**

Single spray and high torque turbine and fixed connection. The turbine is lightweight, easy to clean.

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OPZIONALE

POLARIS

Through a regulation system it's possible to adjust the colour temperature from 4.200° K to 6.000° K in order to improve the contrast on soft tissues.

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10059847

ISO 13485
0152.2020

*"All BTC Medical Equipment is designed,
developed and manufactured in Italy"*

BTC Srl

via Colombarotto, 34
40026 Imola (BO)
Tel. +39 0542 643664